## Plastic Bracket Conditioner (10cc)

REF 470-170

### INSTRUCTIONS FOR USE

#### IMPORTANT: KEEP REFRIGERATED

This Plastic Appliance Conditioner creates a chemical bond between any plastic bracket or acrylic appliance and the bonding cement. It can also be used to condition a composite restoration prior to bonding, or to rebond a bracket impregnated with adhesive.

#### BONDING PLASTIC APPLIANCES:

Use a brush or sponge pellet and apply a thin, uniform coat of Plastic Bracket Conditioner to the plastic surface to be bonded. Wait at least 60 seconds before applying bonding cement. (If more than 10 minutes elapses, re-apply conditioner.)

#### BONDING TO A COMPOSITE RESTORATION:

Use a green stone to lightly rough-up the composite to remove the shine. Clean area with air syringe. Using a brush or sponge pellet and apply a thin, uniform coat of Plastic Bracket Conditioner and allow to dry. If bonding cement is used, apply sealant to the conditioned composite surface and to the cement of the bracket. Place bracket on tooth and position.

#### REBONDING A BRACKET:

Use a bur to roughen the hardened adhesive on the bracket base. With a sponge pellet, apply a thin, uniform coat of Plastic Bracket Conditioner and allow to dry. Apply a thin coat of sealant to the conditioned bracket base. Next, apply sealant to the etched enamel, then apply paste to the bracket, place bracket on tooth and position.

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Ortho Organizers, Inc.
mdi Europa GmbH