Distributed By:

Ortho Organizers, Inc.
1822 Aston Avenue
Carlsbad, CA 92008
USA
Tel: 800.547.2000, +1.760.448.8600
Fax: +1.800.888.7244
USASales@OrthoOrganizers.com
IntlOrders@OrthoOrganizers.com

Visit the *Ortho Organizers* Web site at
www.OrthoOrganizers.com

EC REP

mdi Europa GmbH
Wittekamp 30
D-30163
Hannover, Germany
49 511 39 08 95 30

Advantage™

No-Mix Direct Bonding Adhesive

REF 480-000, 480-000S, 480-000SP,
480-001, 480-020

INSTRUCTIONS FOR USE

ADVANTAGE NO-MIX DIRECT BONDING ADHESIVE
REF 480-000, 480-000S, 480-000SP, 480-001, 480-020
INSTRUCTIONS FOR USE

- Prepare teeth with a non-fluoride, oil-free prophyl paste. Flush thoroughly with water.
- Isolate teeth with cotton rolls or rubber dam.
- Air dry completely with an oil/moisture-free air source.
- Apply one or two drops of etchant onto pad. Using a cotton pellet or brush, dab etching agent onto teeth to be bonded.
- Allow to remain for 60 seconds. Try to avoid contact with gingival tissue.
- Thoroughly rinse the etching agent from the enamel surface with excessive amounts of water to remove all traces of acid (do not wipe etching agent off the enamel surface).
- Air dry thoroughly using oil-and-moisture-free air source. Teeth should have a dull, white frosted appearance.
- Apply a thin uniform coating of primer onto each tooth surface to be bonded.
- Next, apply a thin coating of primer to the underside of each bracket base.
- Adhesive Bonding
 - Apply a small amount of adhesive paste onto the bracket base and place on tooth. In 30 seconds, position the bracket to desired angulation and press firmly.
 - Excess adhesive from around the bracket base can be easily removed after all brackets are in place.
 - Archwires can be placed 5 minutes after bonding of the last bracket.
- Indirect Bonding
 - Proply and etch tooth as prescribed. Place adhesive in a syringe, prime all the teeth to be bonded and all bracket bases in the tray. Using the syringe, dispense a small amount of paste onto each primed bracket base in the tray. Place tray in mouth, wait three minutes and peel tray material away from tooth surface.
- Storage and Use
 - This system is designed to be used at room temperature of approximately 21° to 24°C or 70° to 75°F. Shelf life at room temperature is one year for unopened kits and 90 days for opened kits when caps from jars, bottles, syringes and tips are replaced immediately after dispensing.
 - Note: Although Advantage is hypo-allergenic, prolonged skin contact should be avoided.
 - For best results use minimum primer and minimum paste. Excess of primer or paste will cause bracket flotation.
 - Warning: Etchant contains a 37% Phosphoric Acid solution which can be harmful to skin or eyes. In case of contact, flush with water and seek medical assistance for appropriate treatment.

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ADVANTAGE NO-MIX DIRECT BONDING ADHESIVE

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