

# Photographic Mirrors (#631-090A through 631-090G)

## CLEANING & STERILIZATION INSTRUCTIONS

The recommended methods are steam autoclave or dry heat sterilization.

### INSTRUCTIONS FOR BOTH AUTOCLAVE AND DRY HEAT STERILIZATION:

1. Ultrasonic Cleaning: Use standard ultrasonic cleaner and solution, taking care to prevent the mirror from contacting any other product or metal cassette tray.
2. Hand Wash: Hand wash the mirror with non-abrasive material (soft tissue or gauze).
3. Sterilization: After ultrasonic cleaning and hand washing the photographic mirrors, prior to sterilization, we strongly recommend to pat the mirror dry with a gauze sponge or lint free surgical towel. This will eliminate any water spots during sterilization. Wrap the mirror in a surgical towel, if you choose, when you are autoclaving. If you choose to dry heat sterilize the mirror you can wrap this product, making sure the mirror does not come in contact with any other products and/or metal cassette trays.

Chemical (Cold) Sterilants - In regards to chemical sterilants, there is such a large variety of this particular product in both the orthodontic and dental field. It is difficult for Ortho Organizers to recommend one particular cold sterilant for chemical sterilization. Cold sterilants in 90% of the procedures have proven to be acceptable. Keep in mind that problems do arise with cold sterilants, such as streaks or pitting on the mirror. If you choose to go with cold sterilization, we recommend you contact the manufacturer or experiment with only one mirror, to see the final results.




#631-090A



#631-090B




#631-090C

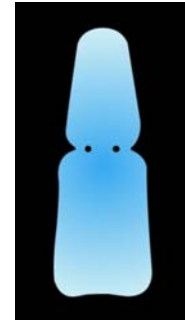


#631-090D



#631-090E

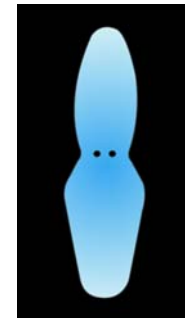

#631-090F

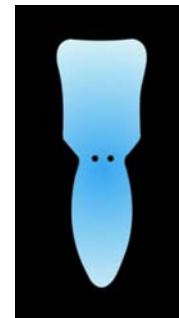

#631-090G



Certificate  
#FM 35078

OMA Member  
Orthodontic Manufacturers  
Association



©2008 Ortho Organizers Mk183.p65 Rev A



Ortho Organizers  
US Toll Free: 1.800.547.2000  
Phone: 760.448.8600  
1822 Aston Avenue  
Carlsbad, CA 92008

Authorized European Representative:  
mdi Europa GmbH  
Witekamp 30, D-30163 Hanover  
Germany Phone: 49 511 39 08 95 30