


Reverse Pull Adjustable Facemask

REF 426-055, -056, -057, -058

Instructions for Use

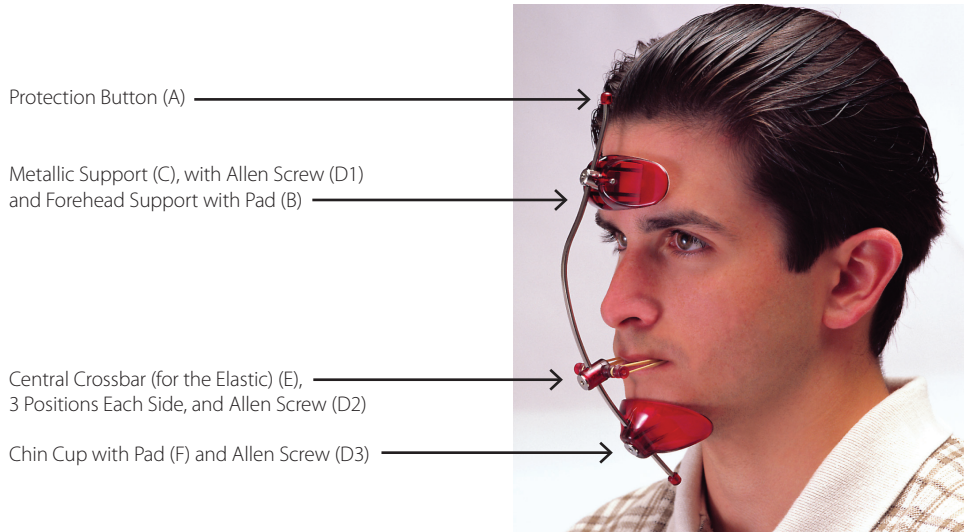

CHARACTERISTICS

The Reverse Pull Adjustable Facemask is made of stainless steel, plastic and anti-allergic pads (2 square pads in each box). The Facemask has been designed for patient comfort while sleeping or speaking with a total weight of 1.86 oz.

HOW TO USE

1. Adjust Chin Cup (F) and tighten Allen Screw (D3).
2. Adjust Forehead Support (B) and tighten Allen Screw (D1).
3. Slide Central Crossbar (E) to the desired position and tighten Allen Screw (D2).
4. Place elastic on bracket or wire hook and stretch forward to place on Crossbar.

We recommend the following elastics:

½" elastics 8 oz. (catalog number #408-073)

½" elastics 16 oz. (catalog number #408-074)



Ortho Organizers, Inc.
1822 Aston Avenue
Carlsbad, CA 92008
USA

Tel: 800.547.2000, +(1) 760 448 8600
Fax: 800.888.7244, +(1) 760 448 8607
USASales@OrthoOrganizers.com
IntlOrders@OrthoOrganizers.com

Visit the Ortho Organizers' Web site at OrthoOrganizers.com



mdi Europa GmbH
Langenhagener Str. 71
30855 Langenhagen, Germany
49 511 39 08 9530



OMA Member
Orthodontic Manufacturers
Association


Made
in the
USA

INS136A
Rev A

Reverse Pull Adjustable Facemask

REF 426-055, -056, -057, -058

Instructions for Use

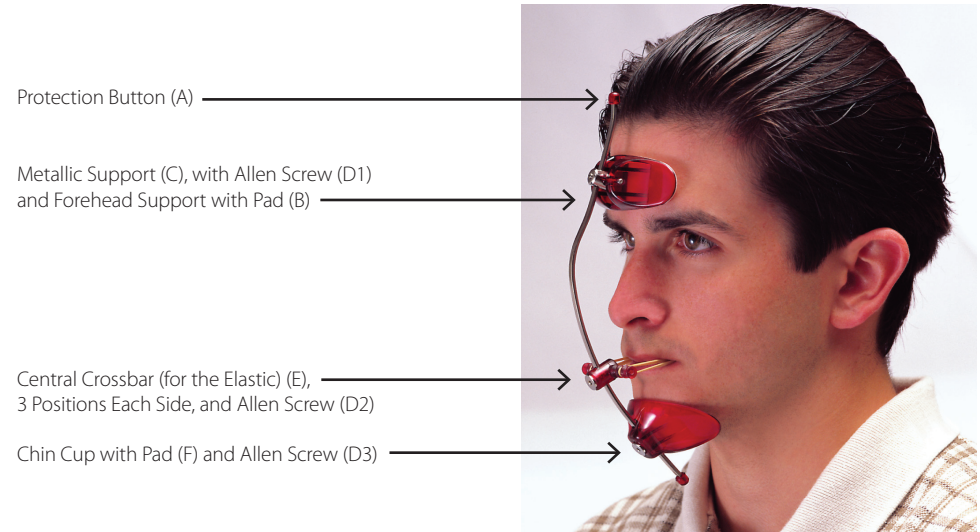

CHARACTERISTICS

The Reverse Pull Adjustable Facemask is made of stainless steel, plastic and anti-allergic pads (2 square pads in each box). The Facemask has been designed for patient comfort while sleeping or speaking with a total weight of 1.86 oz.

HOW TO USE

1. Adjust Chin Cup (F) and tighten Allen Screw (D3).
2. Adjust Forehead Support (B) and tighten Allen Screw (D1).
3. Slide Central Crossbar (E) to the desired position and tighten Allen Screw (D2).
4. Place elastic on bracket or wire hook and stretch forward to place on Crossbar.

We recommend the following elastics:

½" elastics 8 oz. (catalog number #408-073)

½" elastics 16 oz. (catalog number #408-074)



Ortho Organizers, Inc.
1822 Aston Avenue
Carlsbad, CA 92008
USA

Tel: 800.547.2000, +(1) 760 448 8600
Fax: 800.888.7244, +(1) 760 448 8607
USASales@OrthoOrganizers.com
IntlOrders@OrthoOrganizers.com

Visit the Ortho Organizers' Web site at OrthoOrganizers.com



mdi Europa GmbH
Langenhagener Str. 71
30855 Langenhagen, Germany
49 511 39 08 9530



OMA Member
Orthodontic Manufacturers
Association

Made
in the
USA

INS136A
Rev A