Distributed by:

Ortho Organizers, Inc.
1822 Aston Avenue
Carlsbad, CA 92008
USA

Tel: 800.547.2000, +(1) 760 448 8600
Fax: 800.888.7244, +(1) 760 448 8607
USASales@OrthoOrganizers.com
IntlOrders@OrthoOrganizers.com

Visit the *Ortho Organizers'* Web site at
OrthoOrganizers.com

mdi Europa GmbH

Langenhagener Str. 71
30855 Langenhagen, Germany
49 511 39 08 9530

OMA Member
Orthodontic Manufacturers
Association

**Made
in the
USA**

INS19A
Rev E

Illuminate™ Light-Cure™ Adhesive

(with Fluoride, contient du fluor, mit Fluorid,
con fluoro, con flúor)

REF 471-050, 471-053, 471-054

INSTRUCTIONS FOR USE

TOOTH PREPARATION

- Prepare teeth with a non-fluoride/oil-free prophylactic paste.
- Isolate teeth with cotton rolls or rubber dam.
- Air dry completely with an oil/moisture-free air source.

ETCHING SURFACES

- Apply Etchant to surfaces of teeth with cotton pellet or brush. Allow to remain for 15 seconds. On primary teeth and highly mineralized teeth, approximately 2 minutes etch is recommended.
- Flush area completely. Do not allow patient to rinse entire mouth or permit saliva to touch etched enamel surfaces.
- Air dry etched surfaces using oil/moisture-free air source. The etched surfaces should have a dull, whitish frosted appearance. If not, repeat process allowing etchant to remain on teeth for an additional 15 seconds.

PRIMING SURFACES

- Apply a thin uniform coating of Light Cure Orthodontic Bonding Resin onto each tooth surface to be bonded.
- Thin the resin coating on each tooth with a gentle stream of oil/moisture-free air. Apply thin coating of primer to the underside of each bracket base.

ADHESIVE BONDING

- Dispense adhesive paste onto the bracket base. Use sparingly.
- After applying the adhesive, lightly place the bracket onto tooth surface immediately. Gently remove excess adhesive around bracket base.
- Expose adhesive to curing light allowing 10-15 seconds for transparent brackets (20-30 seconds for metal brackets) as a distal edge of approximately 5mm. When using metal brackets, shine the light on the adhesive either from the mesial and distal edges or the occlusal and gingival edges, 10-15 seconds for each edge.
- Archwires can be placed immediately after bonding of the last bracket.

WARNING

- Adhesive paste and bonding resin contains Bis-GMA, an ingredient that in some people can cause an allergic reaction or result in skin or tissue irritation. Avoid contact with the adhesive paste and bonding resin. If contact occurs, wash immediately with soap and warm water. Improper use may result in allergic reaction or skin or tissue irritation. In which case, discontinue use of the product.
- Etchant contains 37% phosphoric acid solution which is harmful to skin or eyes. Avoid contact of etchant to soft tissue or dentin. In case of contact with eyes or skin, immediately flush with water and seek medical assistance for appropriate treatment.
- Do not store materials in proximity to eugenol-containing products. Store at room temperature away from intense light or elevated temperatures.

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