Distributed by:

Ortho Organizers, Inc.
1822 Aston Avenue
Carlsbad, CA 92008
USA
Tel: 800.547.2000, +(1) 760 448 8600
Fax: 800.888.7244, +(1) 760 448 8607
USASales@OrthoOrganizers.com
IntlOrders@OrthoOrganizers.com

Visit the *Ortho Organizers'* Web site at
OrthoOrganizers.com

mdi Europa GmbH
Langenhagener Str. 71
30855 Langenhagen, Germany
49 511 39 08 9530

OMA Member
Orthodontic Manufacturers
Association

**Made
in the
USA**

INS20A
Rev C

Advantage™ No-Mix Direct Bonding Adhesive

**REF 480-000, 480-000S, 480-000SP,
480-001, 480-020**

INSTRUCTIONS FOR USE

Warning: Etchant contains a 37% Phosphoric Acid solution which can be harmful to skin or eyes. In case of contact, flush with water and seek medical assistance for appropriate treatment.

Note: Although Advantage is hypo-allergenic, prolonged skin contact should be avoided. For best results use minimum primer and minimum paste. Excess of primer or paste will cause bracket flotation.

Storage and Use
This system is designed to be used at room temperature of approximately 21° to 24°C or 70° to 75°F. Shelf life at room temperature is one year for unopened kits and 90 days for opened kits when caps from jars, bottles, syringes and tips are replaced immediately after dispensing.

Indirect Bonding
Apply a thin uniform coating of primer onto each tooth surface to be bonded. Next, apply a thin coating of primer to the underside of each bracket base. Position the bracket to desired angulation and press firmly. Excess adhesive from around the bracket base can be easily removed after all brackets are in place. Archwires can be placed 5 minutes after bonding of the last bracket.

Adhesive Bonding
Apply a small amount of adhesive paste onto the bracket base and place on tooth. In 30 seconds, position the bracket to desired angulation and press firmly.

Priming Surfaces
Apply a thin uniform coating of primer onto each tooth surface to be bonded. Next, apply a thin coating of primer to the underside of each bracket base.

Etching Surfaces
Apply one or two drops of etchant onto pad. Using a cotton pellet or brush, dab etching agent onto teeth to be bonded. Allow to remain for 60 seconds. Try to avoid contact with gingival tissue. Thoroughly rinse the etching agent from the enamel surface with excessive amounts of water to remove all traces of acid (do not wipe etching agent off the enamel surface). Air dry thoroughly using oil-and-moisture-free air source. Teeth should have a dull, white frosted appearance.

Tooth Preparation
Prepare teeth with a non-fluoride, oil-free prophylax paste. Flush thoroughly with water. Isolate teeth with cotton rolls or rubber dam. Air dry completely with an oil/moisture-free air source.

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