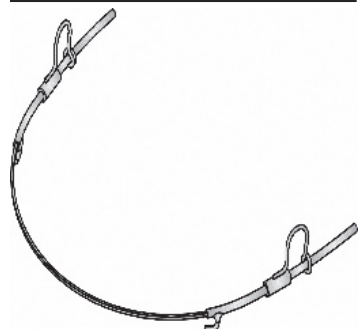
Instructions for use

Multi-Distalizing Arch™


REF 424-150 – 424-170

The *Multi-Distalizing Arch* (MDA) is a multi-function fixed appliance designed for the purpose of moving molars distally and bodily. The MDA can be used either bilaterally or unilaterally, on either the maxillary or mandibular arch.

Procedure*:
Note: It is important to follow these procedures step by step to avoid failures and/or breakage of product. All patients wearing the MDA should be seen every three (3) weeks.

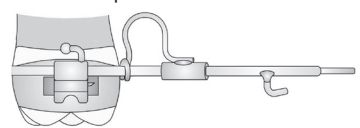
1. **Band both upper and lower first molars with bands that have a .045 gingival buccal tube.** Buccal tubes that are occlusal to the band will result in "tipping" rather than bodily movement during distalization.
2. The anchorage arch (opposite arch of distalization) must be established and have a (minimum of) .018 x .025 Stainless Steel Archwire fully engaged.
3. Two methods can be used to select the proper size of MDA.
 - a. Using a sheet of beeswax or baseplate wax, make a wax impression of the arch that is to be distalized. Select the MDA size that allows the anterior wire to lie passively on the anterior brackets. The distal leg of the omega loop should rest passively 2-3mm from the mesial of the buccal tube (FIG. A).
 - b. Using a flex ruler (supplied in kit; reorder # 120-022), measure around the contour of the arch from the mesial edge of the first molar buccal tube to the mesial edge of the opposite first molar buccal tube. The resulting millimeter measurement is the MDA size required.

4. Compare the patient's arch to the MDA and adapt or recontour appliance if necessary.
5. Using Step Pliers (cat# 201-442), Light Wire Pliers (cat# 201-425), or Bird Beak Pliers (cat# 201-426) place one step-out bend, between 1mm and 2mm, on the left & right mesial to each cuspid (FIG. B). These bends (sometimes referred to as canine eminence bends) add space for the MDA to clear the cuspid & bicuspid brackets. Insufficient clearance of the cuspid & bicuspid brackets could lead to wire fractures.

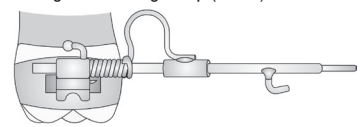6. Determine the length of open coil spring needed by measuring the distance from the mesial edge of the buccal tube to the distal leg of the omega loop and add 2mm. You can use either *Nitanium*® (cat# 100-756) or Stainless Steel Open Coil Spring (cat# 108-105).
7. Thread the measured and cut segments of open coil spring onto the distal legs of the MDA. Seat the MDA into the .045 auxiliary tubes on the first molars. The open coil spring should now be between the mesial edge of the auxiliary tube and the distal leg of the omega loop (FIG. C).

8. Engage the anterior .016 x .022 portion of the MDA into the anterior bracket slots (FIG. D). Ligate the central and lateral incisors with Stainless Steel ligature wire and place a four-unit chain elastic segment connecting the four anterior brackets.

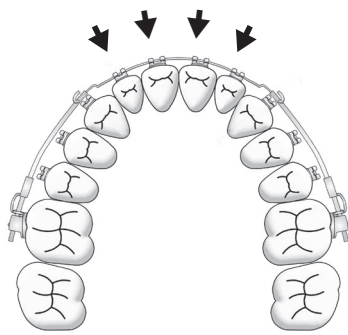
FIG. E

9. Ligate the MDA to the cuspid brackets with Stainless Steel ligature wire (FIG. E). **NOTE:** Ligate MDA wire so it fits snugly on top of cuspid bracket, but do not over tighten. Failure to properly ligate the MDA to the cuspid could result in wire fractures.

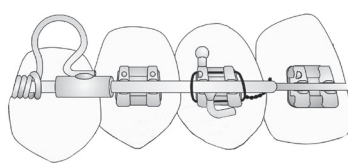
FIG. F

10. Tie a ligature wire from the upper first bicuspid to the Class II hooks of the MDA (FIG. F). This provides anchorage to help limit anterior flaring if the patient fails to follow the class II elastic instructions.

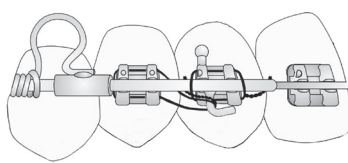
FIG. G

11. Place Class II type intra oral elastics, size Medium 1/4" (cat# 407-041S Amber) or (cat# 407-063S Latex Free). **It is essential that the patient wear the intra oral elastics on a 24-hour schedule, removing only to eat and brush.**

Week 1 - patient should wear three (3) 1/4" Medium Elastics per side of activation.
Week 2 and 3 - patient should wear two (2) 1/4" Medium Elastics per side (FIG. G).

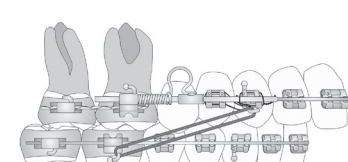
FIG. H

CAUTION: The patient must wear elastics to anchor the MDA. Unwanted flaring of anterior teeth will occur if patients do not wear elastics. For patients who do not comply with Class II elastic wear, it may be necessary to use fixed inter arch mechanics during the distalization phase to prevent any undesirable flaring of the teeth. The *Nitanium* Variable Force Spring (cat# 100-695) is a direct substitute for the class II elastics, and can be ligated into place and worn continuously.

12. The MDA should always be seated on the first molars whether it is used on the upper or lower arch. If second molars are present, it is recommended they be banded and bracketed and incorporated in the Archwire sequence. During the active distalization phase, it is necessary to place rectangular segmental Archwire between the first and second molars. This will prevent the second molars from extruding or being pushed into crossbite. Use braided Stainless Steel Archwire .018 x .025 (cat # 100-373) (FIG. H). ****

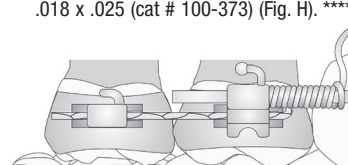
FIG. I

13. On return visits it will be necessary to reactivate the omega loop so that it will continue to compress the coil spring by 2mm. This can be accomplished using Tweed Loop (optical) Pliers (cat# 201-406). Compress the top of the omega loop until the distal leg moves back against the buccal tube, compressing the spring (FIG. I). After each reactivation, it will be necessary for the patient to return to the elastic wearing schedule as explained in step 10.
NOTE: 2mm of overtreatment is necessary for natural rebound once the MDA has been removed.

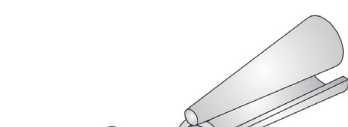
FIG. J

14. On return visits it will be necessary to reactivate the omega loop so that it will continue to compress the coil spring by 2mm. This can be accomplished using Tweed Loop (optical) Pliers (cat# 201-406). Compress the top of the omega loop until the distal leg moves back against the buccal tube, compressing the spring (FIG. I). After each reactivation, it will be necessary for the patient to return to the elastic wearing schedule as explained in step 10.
NOTE: 2mm of overtreatment is necessary for natural rebound once the MDA has been removed.

***This procedure developed in consultation with Dr. Gerety.**

****Dr. Jackson's technique** – Open coil segment should be 6mm in length. When Omega is compressed, it creates equal force bilaterally.


*****Dr. Jackson's technique** – Use 1/4" Light Elastics (cat# 407-040S).

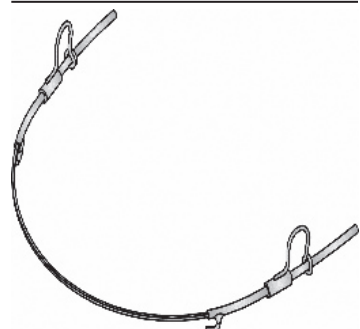******Dr. Jackson's technique** - When second molar present, take at least .016 x .022 Stainless Steel sectional Archwire (usually the last Archwire prior to placing MDA).

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Multi-Distalizing Arch is a registered trademark of Ortho Organizers, Inc.
Ortho Organizers, Inc.
mdi Europa GmbH