
Instructions for use

Ortho Organizers® Twin Force® Bite Corrector Anchor Wire Appliance

REF 424-210Ti, 424-211Ti

Twin Force Bite Corrector Anchor Wire Components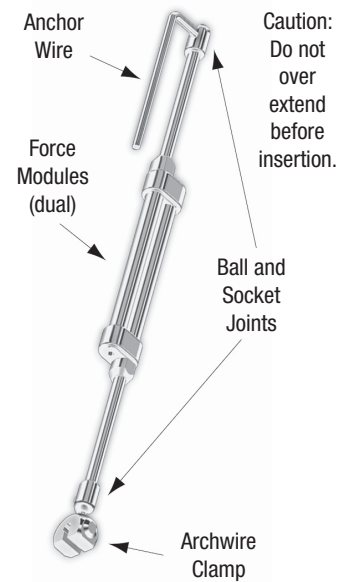

Sizing

The *Twin Force* Bite Corrector Anchor Wire appliance has new size measuring for an even better fit. Please follow these directions for best results. **WARNING:** Take care to select the correct size of *Twin Force* Bite Corrector. Using an incorrect size might lead to breakage and/or complications.


1. Diagnose patient; facially, skeletally, dentally and functionally (condylar position).
2. Begin with a fully pre-programmed appliance like *Ortho Organizers' Elite® Opti-MIM®* Bracket System.
3. Align maxillary dentition to be normal as it relates to the maxilla.
4. Align mandibular dentition to be normal as it relates to the mandible.
5. Lower arch should be flat, bite sufficiently open; a lower lingual arch is recommended.
6. Two sizes of the *Twin Force* Bite Corrector Anchor Wire are available. To determine which size to use, see Figures 1a and 1b.

Class II

- Have the patient bite down.
- Measure from the distal edge of the lower cuspid bracket to the distal end of the upper molar facebow tube (Figure 1a).
- See the chart below for measurement ranges (A) and part numbers.

"A" Measurement	
Minimum to Maximum	REF Number
27mm - 36mm	424-211Ti
32mm - 48mm	424-210Ti

Figure 1a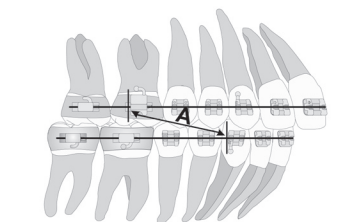


Class III

- Have the patient bite down.
- Measure from the distal edge of the upper cuspid bracket to the distal end of the lower 1st molar lip bumper tube (Figure 1b).
- See the chart below for measurement ranges (A) and part numbers.

"A" Measurement	
Minimum to Maximum	REF Number
27mm - 36mm	424-211Ti
32mm - 48mm	424-210Ti

Figure 1b

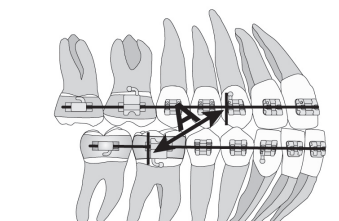PREPARATION (Prior Installation)


1. Place lower fixed lingual arch (Figure 2). Place lower rectangular arch with lingual crown torque of the incisors. Utilize cinch back bends and/or chain elastic molar-to-molar to prevent spacing in the arch. Tie in cuspids with wire ligature ties.

Figure 2

2. Band maxillary first molars with convertible buccal tubes. (Figure 3). Optional: place lingual sheath on molar band, as these can be used later if transpalatal bar or molar rotator is required for expansion or constriction. Place rectangular wire with bite opening curve. Utilize cinch back bends and/or chain elastics to prevent spacing in the arch. Tie-in cuspids with wire ligature ties.


Figure 3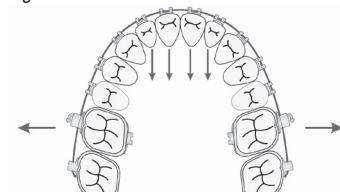

INSTALLATION


1. Place archwire clamp distal to cuspid bracket (Figure 4) and tighten allen screw with *Twin Force* wrench. Cinch back the archwire and/or chain elastic molar to molar to prevent spacing in the arch. The archwire clamp can face occlusal or gingival, it is your preference. You can tie it into the facebow tube first and lock it on the lower archwire second if desired (#4).

Figure 4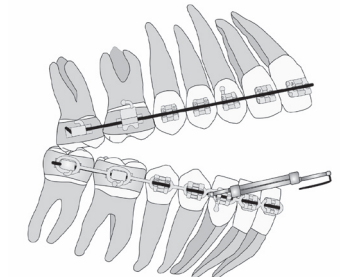


2. Put the anchor wire through the distal end of the facebow tube until the 90 degree bend touches the distal end of the facebow tube (Figure 5). On the mesial end, bend the anchor wire gingival, then distal so it goes underneath the hook and cut off excess. Lock in by pressing the hook towards the tooth surface.

Figure 5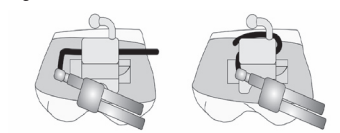

3. Check patient with mouth open and closed. Visually inspect the appliance after installation to ensure it is not fully compressed (Figures 6a & 6b when the patient bites down). **WARNING:** If appliance is fully compressed (Figure 6b), a smaller size is required (order 424-211Ti). If you are already using the 424-211Ti, you should consider other treatment options. Check lateral movement. **WARNING:** Although the appliance allows lateral excursion, instruct the patient not to bite down with a severe lateral excursion as this may break the appliance. Check to see if patient is biting on the appliance, adjust if necessary. A soft diet is recommended for the first 2-3 days, and a prophylactic such as Acetaminophen or Ibuprofen for discomfort.

Figure 6a

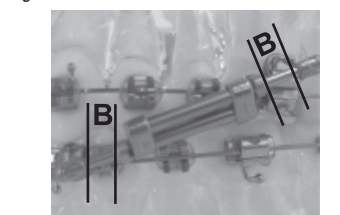Appliance has space (B) between the force module and ball and socket - the appliance is the correct size.

Figure 6b

Force module is fully compressed against the ball and socket - a smaller *Twin Force* Bite Corrector is recommended.


MAINTENANCE

1. See patient in one week, then once a month, minimum.
2. Avoid spacing as the maxillary arch moves distal, and the mandibular arch moves mesial (reverse for Class III).
3. Expect approximately 1-2mm of movement per month, in late mixed dentition or early adult. Most movement is orthodontic, but there is usually 0-2mm of orthopedic changes.
4. Wire tie figure eights for maximum anchorage.

CLASS III CORRECTION

For Class III Correction, lock on upper arch distal to the cuspids and put anchor wire end in a lip bumper tube (LBT) on the lower arch. At times you may need anterior elastics to prevent the bite from opening.

Figure 7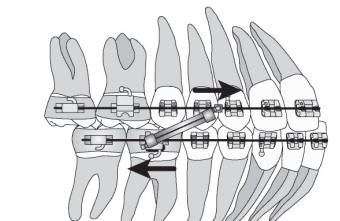

RETENTION (After Appliance Removal)

Use box or vertical elastics to help establish a good buccal occlusion to maintain lower arch stability. *Bite Guide™* bite corrector, #430-005, may also be utilized for anterior guidance.


TIPS

1. A rectangular stainless steel archwire is recommended for both arches for control. (Minimum .016 x .022 for .018 archwire slots and .017 x .025 for .022 archwire slots).
2. Wire tie figure eights in the 5 x 5 for increased control if needed (ligature wire .010 or .012).
3. Use the *Twin Force* Bite Corrector until the anteriors are edge to edge before removing the appliance (you will realize 1-2mm relapse).
4. Locks may be rotated to face gingival or occlusal, whichever is easier.

5. Use cold sterilization only for *Twin Force* wrench.

6. Caution: Do not over extend before insertion. To maintain maxillary molar relationship, use *Ortho Organizers' molar rotator* or transpalatal bar. You can also expand or constrict the molars with the molar rotator (Figure 8).

Figure 8

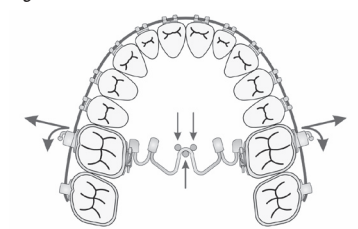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Twin Force is a registered trademark of Ortho Organizers, Inc.
Ortho Organizers, Inc.
mdí Europa GmbH